

# Swab-N-Go Home Test COVID-19 Ag

**For Emergency Use Authorization (EUA) only  
For in vitro diagnostic use**



Immunostics' Swab-N-Go Home Test COVID-19 Ag is a lateral flow immunoassay intended for the qualitative detection of nucleocapsid protein antigens from SARS-CoV-2.

## Benefits

- Results in 15 minutes
- Nasal swab with minimum discomfort
- For use with or without symptoms
- At home test for adults and kids aged 2 years and above

## Reliability


- Positive Percentage Agreement: 87.3%
- Negative Percentage Agreement: 100%

## Convenience


- 20 months shelf life
- Simple 2T packaging
- Usability of RAPIDCHECK app for iOS & Android
- Room temperature storage (2-30 °C)

## RAPIDCHECK application

- Access simple and animated step-by-step guided instructions
- Test result management
- Available on iOS (14.2 or later version) & Android (10 or later version)

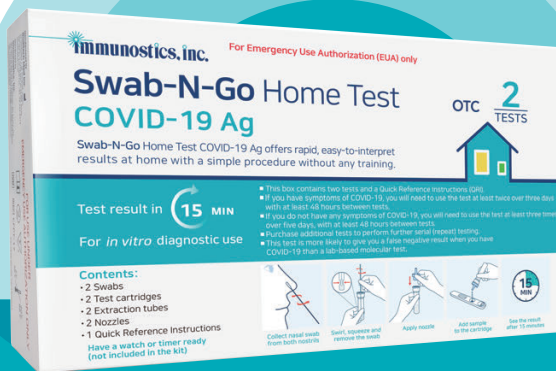

Scan the QR code  
and open the App.

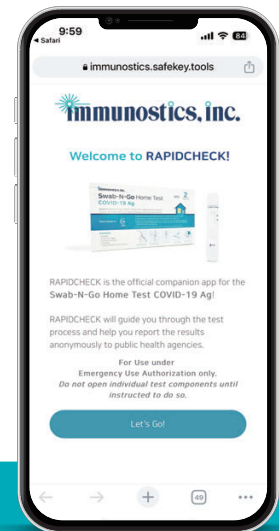


- In the USA, this product has not been FDA cleared or approved; but has been authorized by FDA under an EUA;
- This product has been authorized only for the detection of proteins from SARS-CoV-2, not for any other viruses or pathogens; and,
- The emergency use of this product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of in vitro diagnostics for detection and/or diagnosis of COVID-19 under Section 564(b)(1) of the Federal Food, Drug and Cosmetic Act, 21 U.S.C. § 360bbb-3(b)(1), unless the declaration is terminated, or authorization is revoked sooner.
- For more information on EUAs please visit:  
<https://www.fda.gov/emergency-preparedness-and-response/mcm-legal-regulatory-and-policy-framework/emergency-use-authorization>
- For the most up to date information on COVID-19, please visit: [www.cdc.gov/COVID19](http://www.cdc.gov/COVID19)

# Swab-N-Go Home Test COVID-19 Ag



## Swab-N-Go Home test COVID-19 Ag Test Procedure

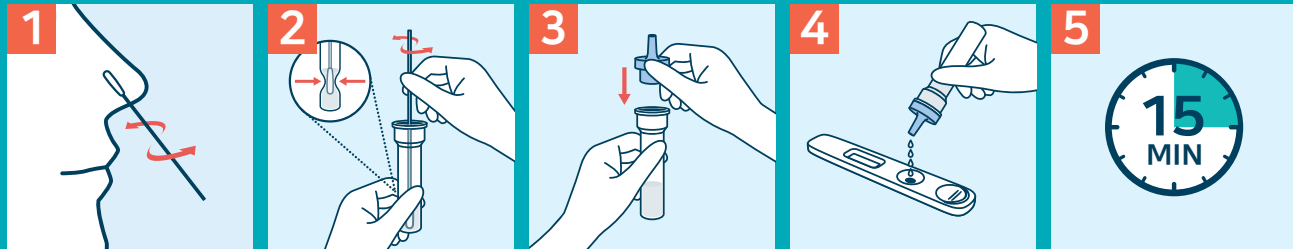

**1**  
Collect nasal swab from both nostrils


**2**  
Swirl, squeeze and remove the swab

**3**  
Apply nozzle

**4**  
Add sample to the cartridge

**5**  
See the result after 15 minutes

## Interpretation of result

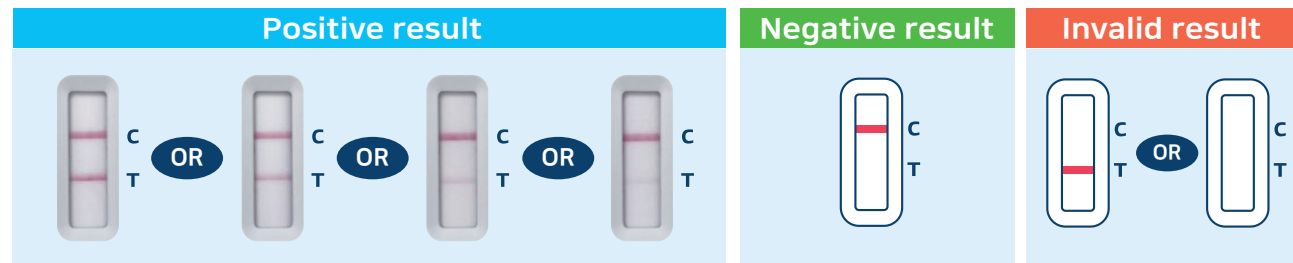

## Ordering Information

REF CFPT-17

### Swab-N-Go Home test COVID-19 Ag 2T (Cat.#: CFPT-17)

#### Kit Includes:

• Swabs	2 ea
• Test cartridge	2 ea
• Extraction tubes	2 ea
• Nozzles	2 ea
• Quick Reference Instruction	1 ea



\* Have a watch or timer ready (not included in the kit)

\* Available for bulk purchase. For further details, please contact us using the information provided below